


UPGRADE
PROTONPEN
MICRONEEDLES
for enhanced
microneedling
results



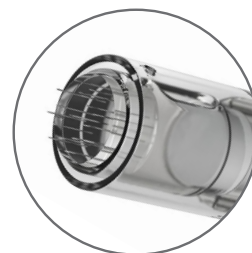

We are excited to announce an upgrade to our microneedling treatments with a custom-designed cartridge for our PROTONPEN microneedles.

This improvement raises the positioning of the needles, optimising the precision and effectiveness of the treatment.

Please note that while you should continue to follow the recommended needle depth chart for specific areas of the face, neck, and décolletage, the increased stimulation and intensity of the upgraded needles may require slight adjustments to the needle depth based on the individual skin type being treated.

We are confident that this upgrade will significantly enhance treatment outcomes and work synergistically with our active resurfacing products to deliver optimal treatment results.

	REGION	FRAGILE SKIN	RESILIENT SKIN
	Forehead	0.25 – 0.5mm	0.5mm
	Between eyebrows	0.25 – 0.5mm	0.5mm
	Nose	0.25mm	0.5mm
	Eye area	0.25mm	0.25mm
	Cheek bone	0.5mm	0.5 – 0.75mm
	Cheek	0.5 – 0.75mm	0.5 – 1mm
	Lips	0.25mm	0.25mm
	Neck & décolleté	0.25 – 0.5mm	0.25 – 0.5mm
	Superficial scars	1.5mm	1.5mm